# Reinigung & Wartung Cleaning & Maintenance

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- Basically it is not a problem cleaning your K-COM knee brace with warm soap water; it is however then recommended to protect the hinges against the water.
- After using your K-COM in salt water you should then rinse it with fresh water.
- After a beach holiday your K-COM needs to be serviced to avoid sand grinding the hinges.
- Your K-COM should generally be serviced 1-2 times a year.
- Should your leg change in volume or any other problems arise please contact us as soon as possible.
- We also offer breathable undergarments to wear with your K-COM.

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## Safety instructions:

The K-COM knee brace can reduce the risk of injury inflicted by an impact or contortion of the knee. A general protection against injuries however cannot be guaranteed. Warranty and liability are rendered void by:

- Not using the brace as intended
- Improper maintenance
- Modifying the product without our approval
- Any kinds of accidents involving the brace
- No liability applies for personal injuries and damage to property such as other circumstances for which the manufacturer cannot be made responsible
- Force majeure

Precautionary measures during sports and leisure activities greatly increase your safety and reduce the risk of injury.

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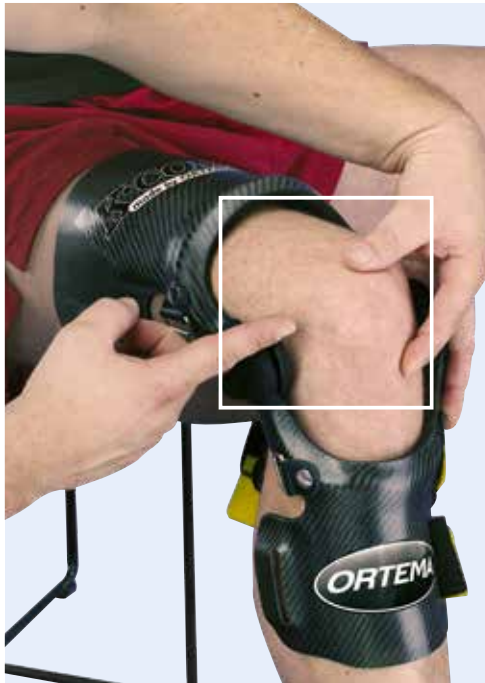
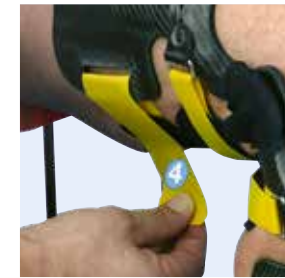
The K-COM is donned while slightly bending your knee. Hinges are aligned to the middle of the knee cap.

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The straps should be closed regarding points 1 to 4. Especially the calf strap closest to the knee should be fastened as tight as possible.

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Close the wider strap on the calf ensuring it has full contact with the skin.

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The straps should always be looped on the outside of the brace.

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After closing the fourth strap the brace should fit snugly and evenly when tensing the muscles.

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The order of the straps applies for K-COM, X-PERT, and IPOMAX knee braces alike. **Check fit:** When the hinges of the brace remain at the same height of the knee cap the brace has been applied correctly.

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**Kröger's Centrum für Technische Orthopädie GmbH**